

## Instructions for use – AF SERVO

Thank you for purchasing AF SERVO. AF SERVO is a ready-to-go ankle foot orthosis for foot drop patients, which obviates the fabrication time delays required by customized AFOs, thanks to turning dial lock which allows on-the-fly adjustment by user.

The use of the AF SERVO's turning dial, instead of a Velcro, allows firm adherence of the orthosis to the user's body shape and provides support and comfort in daily use to a foot-drop symptom patient, while easing tasks such as putting shoes on or taking them off. Additionally, the turning dial adjustment ensures comfortable fit and does not allow loosening, thus preventing secondary damage.

### Intended Use

AF SERVO is an ankle foot orthosis and is intended to be used for mild foot drop symptoms due to injuries such as stroke, spine or nerve damage, bone fracture, etc. This product must be used only under guidance of a medical professional such as a physical therapist, physician, orthotist or prosthetist. Consult your medical professional if this product is suitable for your symptoms.

### Caution

1. This product is intended to be worn with socks and to be used for mild foot drop symptoms and only under the direction of a medical professional. If you experience any pain, swelling, sensation changes, or any unusual reactions while using this product, consult your medical professional immediately.
2. This product is a supportive orthosis and is not guaranteed to prevent injury.
3. Do not turn the dial counterclockwise, or try to take the dial apart, which may cause damage to the product.
4. This product is hand-washable only, with cold water and a mild detergent. Do not use bleach and do not squeeze.
5. Dry in the shade, and do not use dryer or exposing to the sun or heat.

### How to Wear



1. Pull the dial lock until it clicks, and then appropriately open the top cover.



2. Slide in your foot into the AF SERVO's body frame.



3. Fit the AF SERVO on your foot and push the dial lock firmly until it clicks.



4. Turn the dial lock clockwise to adjust the AF SERVO to your foot shape.

### How to Wear with a Shoe



1. Place AF SERVO into the shoe and pull out the dial until it clicks. Then open the top cover.



2. Slide in your foot into the AF SERVO body frame.



3. Adjust the top cover position to make it comfortable, and push the dial to lock firmly until it clicks.



4. Turn the dial lock clockwise to adjust the AF SERVO to your foot shape.